

**Purification of viral RNA from plasma for viral sequencing using the VERSANT kPCR Sample Prep System and the VERSANT Sample Preparation 1.0 Reagent Kit**

This protocol has been adapted by customers for use with the VERSANT® kPCR Sample Prep System and the VERSANT Sample Preparation 1.0 Reagent Kit. This protocol has not been thoroughly tested and optimized by Siemens.

**Introduction**

The VERSANT Sample Preparation 1.0 Reagent Kit is a reagent kit for the isolation of RNA and DNA. The kit can be used for extraction of nucleic acids in conjunction with the VERSANT kPCR Sample Prep system. The VERSANT Sample Preparation 1.0 Reagent Kit includes a lysis reagent which lyses the cells and denatures nucleases while leaving RNA and DNA intact. The released nucleic acids are captured on magnetic beads. Three different wash buffers are used to remove all proteins, nucleases and other contaminants, and then purified nucleic acids are eluted.

**Notes**

Please refer to the following instructional materials for the VERSANT kPCR Sample Prep System before beginning this protocol:

- *VERSANT kPCR Molecular System SP Application Guide (10379670 Rev. A, 2009-06)*
- *VERSANT Stand-alone Sample Prep (kPCR) Operator Checklist (NA0896 Rev. 0)*

**Materials provided in the VERSANT Sample Preparation 1.0 Reagent Kit include:**

<b>Box 1</b> <ul style="list-style-type: none"><li>• Magnetic beads</li><li>• Lysis Buffer</li><li>• Wash Buffer 1</li><li>• Wash Buffer 2</li><li>• Wash Buffer 3</li><li>• Elution Buffer</li></ul>	<b>Box 2</b> <ul style="list-style-type: none"><li>• Proteinase K</li></ul>
---	---

**Additional materials required by this protocol but not included in the VERSANT Sample Preparation 1.0 Reagent Kit:**

- VERSANT kPCR Sample Prep Module

# User Developed Protocol

## Viral Sequencing

### Procedure

1. Switch on VERSANT kPCR Sample Prep module and perform the recommended daily maintenance.  
Load the system with the required consumables including tips, deep well plate and one 96-well PCR plate.
2. Pour the VERSANT Sample Preparation Reagents into the reagent troughs and place them on the VERSANT Sample Prep module.
3. Load the plasma samples onto the sample carrier. When placing the samples in the sample carrier, carefully remove the tube caps.
4. Place the sample carriers on the auto load tray of the VERSANT Sample Prep module.
5. The VERSANT kPCR Sample Prep System is fully automated from this step onward. At the Main Menu Screen:  
Select Protocol #4 (500 µL of liquid sample for extraction / delivers 50 µL eluate).
6. Start run.
7. Run time: Approximately 1½ – 3 hours for extractions of 1 to 96 samples (depending on batch size).
8. Remove the 96-well PCR plate containing the sample extracts.
9. Use 17µl of extracted RNA to perform RT-PCR and DNA sequencing.

It is the user's responsibility to validate the performance of the VERSANT kPCR Sample Prep System and VERSANT Sample Preparation 1.0 Reagent Kit for any particular application, since their performance characteristics have not been validated for any specific use (research, diagnostic, prognostic, or therapeutic). This procedure may be used in clinical diagnostic laboratory systems after the laboratory has validated their complete system as required by CLIA '88 regulations in the U.S. or equivalents in other countries.

Siemens Healthcare Diagnostics, the leading clinical diagnostics company, is committed to providing clinicians with the vital information they need for the accurate diagnosis, treatment and monitoring of patients. Our comprehensive portfolio of performance-driven systems, unmatched menu offering and IT solutions, in conjunction with highly responsive service, is designed to streamline workflow, enhance operational efficiency and support improved patient care.

VERSANT, and all associated marks are trademarks of Siemens Healthcare Diagnostics Inc. All other trademarks and brands are the property of their respective owners.

Product availability may vary from country to country and is subject to varying regulatory requirements. Please contact your local representative for availability.