

## C-Peptid

- **Assay Format:** IMMULITE (LKPEP), IMMULITE 2000 (L2KPEP), IMMULITE 2500 (L5KPEP)
- **Messbereich:** bis 15 ng/ml (WHO 1st IRP 84/510)
- **Analyt. Sensitivität:** 0,05 ng/ml (IMMULITE , IMMULITE 2000 , IMMULITE 2500)
- **Umrechnungsfaktor:** ng/ml x 331 → pmol/l  
ng/ml x 0,331 → nmol/l

Probanden	System	n	Median (ng/ml)	95%-Bereich (ng/ml)	97,5. Perc. (ng/ml)	Lit.
Gesunde, nüchtern	IMMULITE IMMULITE 2000 IMMULITE 2500	136	2,2	0,9 – 7,1	7,1	PILKPEP-1 PIL2KPEP-3 PIL5KPEP-3

### Urin

Probanden	System	n	Median (µg/d)	95%-Bereich (µg/d)	97,5. Perc. (µg/d)	Lit.
Gesunde, Urin	IMMULITE IMMULITE 2000 IMMULITE 2500	82		2,5 - 249 3,6 – 253 2,5 - 232	249 253 232	PILKPEP-1 PIL2KPEP-3 PIL5KPEP-3