

TSH						
<ul style="list-style-type: none"> • Assay Format: IMMULITE 3rd Generation TSH (LKTS), IMMULITE 2000 3rd Generation TSH (L2KTS), IMMULITE 2500 3rd Generation TSH (L5KTS) • IMMULITE rapid TSH (LKRT), IMMULITE 2000 rapid TSH (L2KRT), IMMULITE 2500 rapid TSH (PIL5KRT) • Messbereich: bis 75 µIU/ml (WHO 2nd IRP 80/558) • Analyt. Sensitivität: 0,004 µIU/ml (IMMULITE 3rd Generation TSH; IMMULITE 2000 3rd Generation TSH, IMMULITE 2500 3rd Generation TSH) 0,01 µIU/ml (IMMULITE rapid TSH; IMMULITE 2000 rapid TSH, IMMULITE 2500 rapid TSH) 						
Probanden	System	n	Median (µIU/ml)	95% - Bereich (µIU/ml)	97,5. Perc. (µIU/ml)	Lit.
Euthyreote Erwachsene	IMMULITE IMMULITE 2000 IMMULITE 2500	152	1,3	0,4 – 4,0	4,0	PILKTS-8 PIL2KTS-12 PIL5KTS-3 PILKRT-8 PIL2KRT-12 PILKRT-4
Schwangere	IMMULITE	144	1,1	0,2 – 3,5	3,5	20
Männer 21-30a	IMMULITE	50	1,4	0,5 – 2,9	2,9	10
Männer 31-40		50	1,1	0,3 – 2,2	2,2	
Männer 41-50		50	1,0	0,4 – 3,1	3,1	
Männer 51-60		50	1,0	0,5 – 1,7	1,7	
Männer 61-70a		50	0,9	0,3 – 1,7	1,7	
Männer > 70a		50	1,0	0,4 – 3,0	3,0	
Kinder, 1a	IMMULITE IMMULITE 2000 IMMULITE 2500		2,1	0,4 – 8,6	8,6	PILKTS-8 PIL2KTS-12 PIL5KTS-3
Kinder, 2a			1,8	0,4 – 7,6	7,6	
Kinder, 3a			1,6	0,3 – 6,7	6,7	
Kinder, 4a			1,6	0,3 – 6,3	6,3	
Kinder, 5a			1,6	0,3 – 6,1	6,1	
Kinder, 6a			1,6	0,3 – 6,0	6,0	
Kinder, 7a			1,6	0,3 – 5,8	5,8	
Kinder, 8a			1,6	0,3 – 5,7	5,7	
Kinder, 9a			1,5	0,3 – 5,6	5,6	
Kinder, 10a			1,5	0,4 – 5,5	5,5	
Kinder, 11a			1,5	0,4 – 5,5	5,5	
Kinder, 12a			1,5	0,4 – 5,4	5,4	
Kinder, 1-7d	IMMULITE	45	4,6	1,8 – 9,7	9,7	19
Kinder, 8-15d		40	3,7	1,8 – 8,0	8,0	
Kinder, 16d-3a		86	2,0	0,6 – 4,1	4,1	
Kinder, 4-6a		51	1,6	0,5 – 2,9	2,9	
Kinder, 7-8a		50	1,9	0,8 – 3,5	3,5	
Kinder, 9-10a		71	2,0	0,8 – 3,5	3,5	
Kinder, 11a		45	1,9	0,8 – 3,3	3,3	
Kinder, 12a		35	1,9	0,9 – 3,2	3,2	
Kinder, 13a		46	1,8	0,8 – 3,1	3,1	
Kinder, 14a		62	1,7	0,8 – 2,8	2,8	
Kinder, 15a		88	1,6	0,7 – 2,6	2,6	
Kinder, 16a		71	1,5	0,6 – 2,5	2,5	
Kinder, 17a		52	1,5	0,6 – 2,4	2,4	
Kinder, 18-19a		20	1,4	0,5 – 2,4	2,4	
Kinder, 1d-19a		762	1,7	0,7 – 3,0	3,0	
Jungen, Tanner 1		IMMULITE	28	1,8	0,7 – 3,8	
Jungen, Tanner 2	25		2,0	1,0 – 3,3	3,3	
Jungen, Tanner 3	47		1,8	0,8 – 3,0	3,0	
Jungen, Tanner 4	31		1,8	0,8 – 2,9	2,9	
Jungen, Tanner 5	65		1,7	0,8 – 3,0	3,0	
Mädchen, Tanner 1	IMMULITE	33	1,8	0,7 – 3,7	3,7	19
Mädchen, Tanner 2		25	1,9	0,8 – 3,2	3,2	
Mädchen, Tanner 3		50	1,8	0,8 – 3,1	3,1	
Mädchen, Tanner 4		39	1,8	0,8 – 2,9	2,9	
Mädchen, Tanner 5		38	1,6	0,7 – 2,8	2,8	