

AutoAlign Head: LS

1. Auto Align Head: Landmark Survey
2. Load AA Head study into the measurement queue. The AutoAlign scout will run automatically. This scan is a 3D series, which produces 2 series. 1 with the "Head Basis" boney landmark matrix. And one with the "Head Brain" matrix angled with the AC-PC Line.
3. To ensure proper alignment, open the "Auto Align Information Dialog".
4. Based on positioning preferences, choose one of the AA matrixes from the Auto Align dialog.
5. After verifying your positioning, apply the scan.

