

syngo.via: Manual Mapping of Series to Workflow Layouts

1. With the desired patient assigned to the desired Workflow and the patient opened in the workflow viewer (see document “assigning a workflow” for more information) you will notice that most of the images did not load.
2. Let’s take this opportunity to work with the series navigator. Click the blue arrow on the right hand side of the screen. This will pull up the entire series list.
3. Drag in the desired images into the lower left and right frames. You will need to expand the arrows next to the images and select latest to pull them in.
4. Once you have dragged all of the desired images into the desired segments, click the arrow to close the series browser. The study is now manually mapped to the workflow.

