



Case Study

syngo DynaCT Cardiac

Hypoplastic Left Heart Syndrome

www.siemens.com/healthcare

SIEMENS

Hypoplastic Left Heart Syndrome

Courtesy of PD Dr. P. Ewert and Prof. F. Berger, Dept. of Congenital Heart Diseases, German Heart Institute Berlin

Congenital Heart Disease

Patient History

5-year-old male (20 kg) with Hypoplastic Heart Syndrome; pre-Fontan surgery evaluation.

Diagnosis


syngo DynaCT Cardiac demonstrated visualization of the pulmonary vasculature and the cavopulmonary anastomosis. The 3D images confirmed the stent position within a patent pulmonary artery. The patient was mildly sedated and the *syngo* DynaCT images were acquired during normal respiration.

Protocol


5 sec rotational angiography,
130 projection images.
Injection: 20 cc contrast, 60 cc total volume
(dilution 1:2), injection rate 10 ml/sec,
1 sec X-ray delay.

Comment

This was the first worldwide pediatric case with *syngo* DynaCT Cardiac. The excellent three-dimensional visualization of the pulmonary tree versus the whole lung vessel tree is comparable to conventional CT imaging with significantly reduced radiation exposure. *syngo* DynaCT Cardiac images provided optimal image projections to aid in the successful treatment planning and intervention.

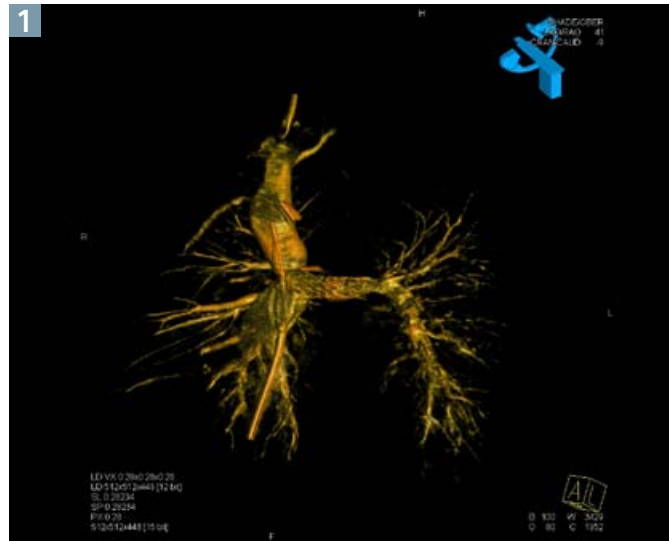

1 Picture: Three-dimensional reconstruction of the superior caval vein, the bidirectional Glenn-anastomosis and the pulmonary arterial vessels. A stent is visible in the central pulmonary arteries. (Ewert, DHZB, German Heart Institute Berlin)

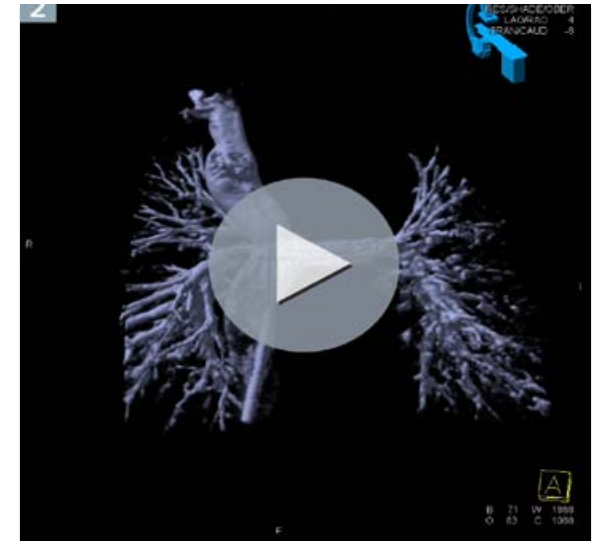

2 Movie clip: The 3D images confirmed the stent position within a patent pulmonary artery. (Ewert, DHZB, German Heart Institute Berlin)

On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this brochure are available through the Siemens sales organization worldwide. Availability and packaging may vary by country and are subject to change without prior notice. Some/ All of the features and products described herein may not be available in the United States or other countries.

The information in this document contains general technical descriptions of specifications and options as well as standard and optional features that do not always have to be present in individual cases.

Siemens reserves the right to modify the design, packaging, specifications and options described herein without prior notice. Please contact your local Siemens sales representative for the most current information.

Global Siemens Headquarters

Siemens AG
Wittelsbacherplatz 2
80333 Muenchen
Germany

In the interest of complying with legal requirements concerning the environmental compatibility of our products (protection of natural resources and waste conservation), we recycle certain components. Using the same extensive quality assurance measures as for factory-new components, we guarantee the quality of these recycled components.

Note: Any technical data contained in this document may vary within defined tolerances. Original images always lose a certain amount of detail when reproduced.

www.siemens.com/medical-accessories

Global Siemens Healthcare Headquarters

Siemens AG
Healthcare Sector
Henkestrasse 127
91052 Erlangen
Germany
Telephone: +49 9131 84-0
www.siemens.com/healthcare

Global Business Unit

Siemens AG
Medical Solutions
Angiography, Fluoroscopic and Radiographic Systems
Siemensstrasse 1
DE-91301 Forchheim
Germany
Telephone +49 9191 18-0
www.siemens.com/healthcare

Legal Manufacturer

Siemens AG
Wittelsbacherplatz 2
DE-80333 Muenchen
Germany

www.siemens.com/healthcare