

SIEMENS MAGNETOM Verio syngo MR B17

\\USER\knee_20min_0001\20min_protocols_NYU_27.05.09\shoulder_4ch_okt2_BLADE_fs_cor_320

TA: 3:14 PAT: Off Voxel size: 0.4x0.4x3.0 mm Rel. SNR: 1.00 SIEMENS: tse

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

Routine

Slice group 1	
Slices	20
Dist. factor	10 %
Position	Isocenter
Orientation	Coronal
Phase enc. dir.	R >> L
Rotation	0.00 deg
Phase oversampling	37.5 %
FoV read	140 mm
FoV phase	100.0 %
Slice thickness	3.0 mm
TR	3500 ms
TE	72 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	SHS

Contrast

MTC	Off
Magn. preparation	None
Flip angle	150 deg
Fat suppr.	Fat sat.
Fat sat. mode	Weak
Water suppr.	None
Restore magn.	Off
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution

Base resolution	320
BLADE coverage	100.0 %
Trajectory	BLADE
Interpolation	Off
PAT mode	None
Matrix Coil Mode	Auto (CP)
Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	Off
B1 filter	Off
Raw filter	Off
Elliptical filter	Off

Geometry

Multi-slice mode	Interleaved
Series	Interleaved
Special sat.	None
Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off
Tim CT mode	Off

System

Body	Off
SHS	On
Positioning mode	FIX
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	H >> F
Coil Combine Mode	Adaptive Combine
AutoAlign	---
Auto Coil Select	Default
Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Coronal
Rotation	0.00 deg
F >> H	140 mm
R >> L	140 mm
A >> P	66 mm

Physio

1st Signal/Mode	None
Dark blood	Off
Resp. control	Off

Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Sequence

Introduction	On
Dimension	2D
Compensate T2 decay	Off
Contrasts	1
Bandwidth	363 Hz/Px
Flow comp.	No
Allowed delay	60 s
Echo spacing	10.3 ms

SIEMENS MAGNETOM Verio syngo MR B17

Define	Turbo factor
Turbo factor	13
Echo trains per slice	54
RF pulse type	Normal
Gradient mode	Fast