

SIEMENS MAGNETOM Product syngo MR B17

\\USER\knee_20min_0002\20min_protocols_NYU_27.05.09\wrist_8ch_ok\pd_tse_fs_tra

TA: 3:45 PAT: Off Voxel size: 0.3x0.3x2.2 mm Rel. SNR: 1.00 SIEMENS: tse

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

Routine

Slice group 1	
Slices	25
Dist. factor	10 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Phase oversampling	30 %
FoV read	80 mm
FoV phase	100.0 %
Slice thickness	2.2 mm
TR	4560 ms
TE	50 ms
Averages	1
Concatenations	1
Filter	Elliptical filter
Coil elements	WR

Contrast

MTC	Off
Magn. preparation	None
Flip angle	150 deg
Fat suppr.	SPAIR
Fat sat. mode	Weak
Water suppr.	None
Restore magn.	Off
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Off

Resolution

Base resolution	320
Phase resolution	100 %
Phase partial Fourier	Allowed
Trajectory	Cartesian
Interpolation	Off
PAT mode	None
Matrix Coil Mode	Auto (CP)
Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	Off
B1 filter	Off
Raw filter	Off

Elliptical filter On
Mode Inplane

Geometry

Multi-slice mode	Interleaved
Series	Interleaved
Special sat.	Parallel H
Gap	10 mm
Thickness	50 mm
Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off
Tim CT mode	Off

System

Body	Off
WR	On
Positioning mode	FIX
MSMA	S - C - T
Sagittal	L >> R
Coronal	A >> P
Transversal	H >> F
Coil Combine Mode	Adaptive Combine
AutoAlign	---
Auto Coil Select	Default
Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
R >> L	80 mm
A >> P	80 mm
F >> H	61 mm

Physio

1st Signal/Mode	None
Dark blood	Off
Resp. control	Off

Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

SIEMENS MAGNETOM Product syngo MR B17

Sequence

Introduction	On
Dimension	2D
Compensate T2 decay	Off
Reduce Motion Sens.	Off
Contrasts	1
Bandwidth	260 Hz/Px
Flow comp.	Slice
Allowed delay	30 s
Echo spacing	12.4 ms

Define	Turbo factor
Turbo factor	9
Echo trains per slice	47
RF pulse type	Normal
Gradient mode	Fast