
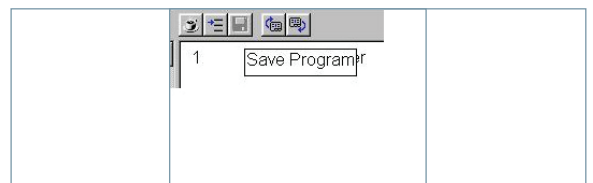
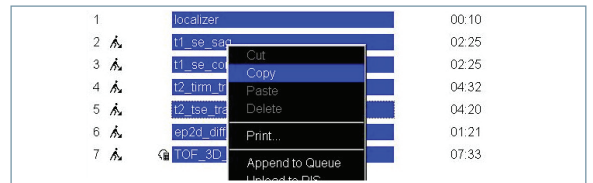
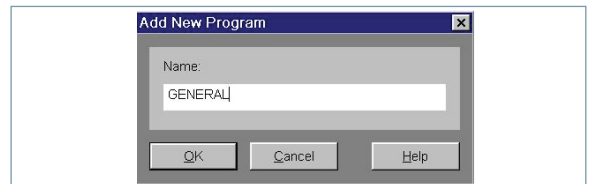
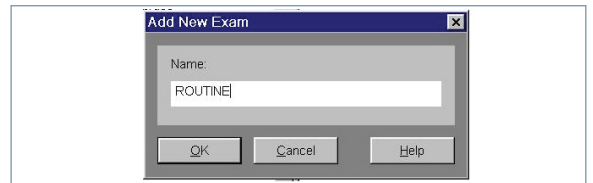


Protocol Building in the Exam Explorer

1. There are three levels of the USER Tree: Region, Exam, and Program. This guide will start by creating a new Region but you can start from any level.
2. Select **USER** on the left side of the Exam Explorer. Right mouse click and select **New**. Name your region and select **OK**.
3. Select the Region you just named, then right mouse click and select **New**. Name your exam and select **OK**.
4. Select the Exam you just named, then right mouse click and select **New**. Name your program and select **OK**.
5. Once you are on the program level, you are now ready to copy and paste sequences into your new program.
6. After you have your program complete with all the required sequences, select the **Save** icon to save this as a new USER program.

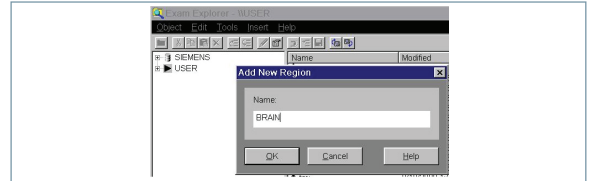

Protocol Building in the Exam Explorer

www.usa.siemens.com/healthcare

SIEMENS