
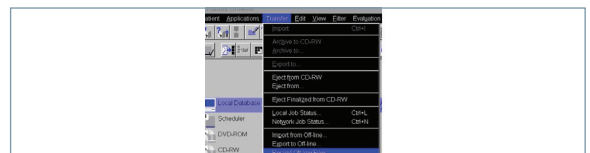
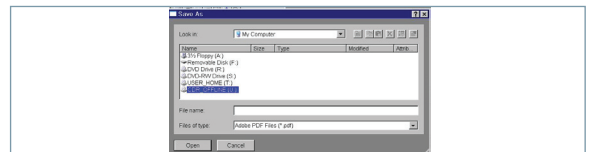
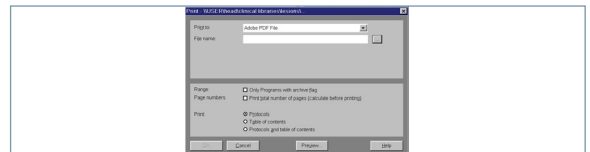
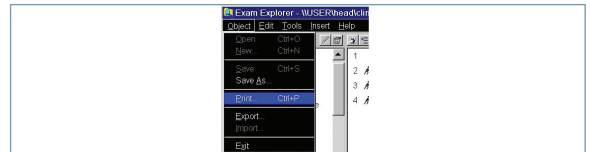


Print Protocols

1. Open the Exam Explorer and select either the Region, Exam, or Program you wish to save as a .pdf.
2. Select the object drop-down menu, then choose print.
3. On the pop-up screen, select to save file as a .pdf file. Then, choose the icon seen on the right with three dots.
4. In the Save As window, Look in: My Computer. Select CDR_Offline. Then, name your file name whatever you wish but make sure that the ending is .pdf as seen. Select **Open**.
5. Then choose **OK** to save.
6. Open the patient browser and choose the transfer drop-down menu, select **local job status**. Delete all jobs that are currently in the local job status.
7. Insert a new CD into the CD-R drive. Then, choose the transfer drop-down menu and record offline files. The CD will eject from the drive when the archive is complete.

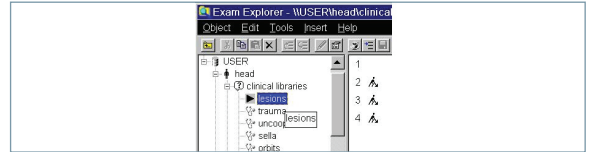

MR Print Protocols

www.usa.siemens.com/healthcare

SIEMENS